Manuale Installazione +
STran**plus**

REV.03/2024

Prefazione

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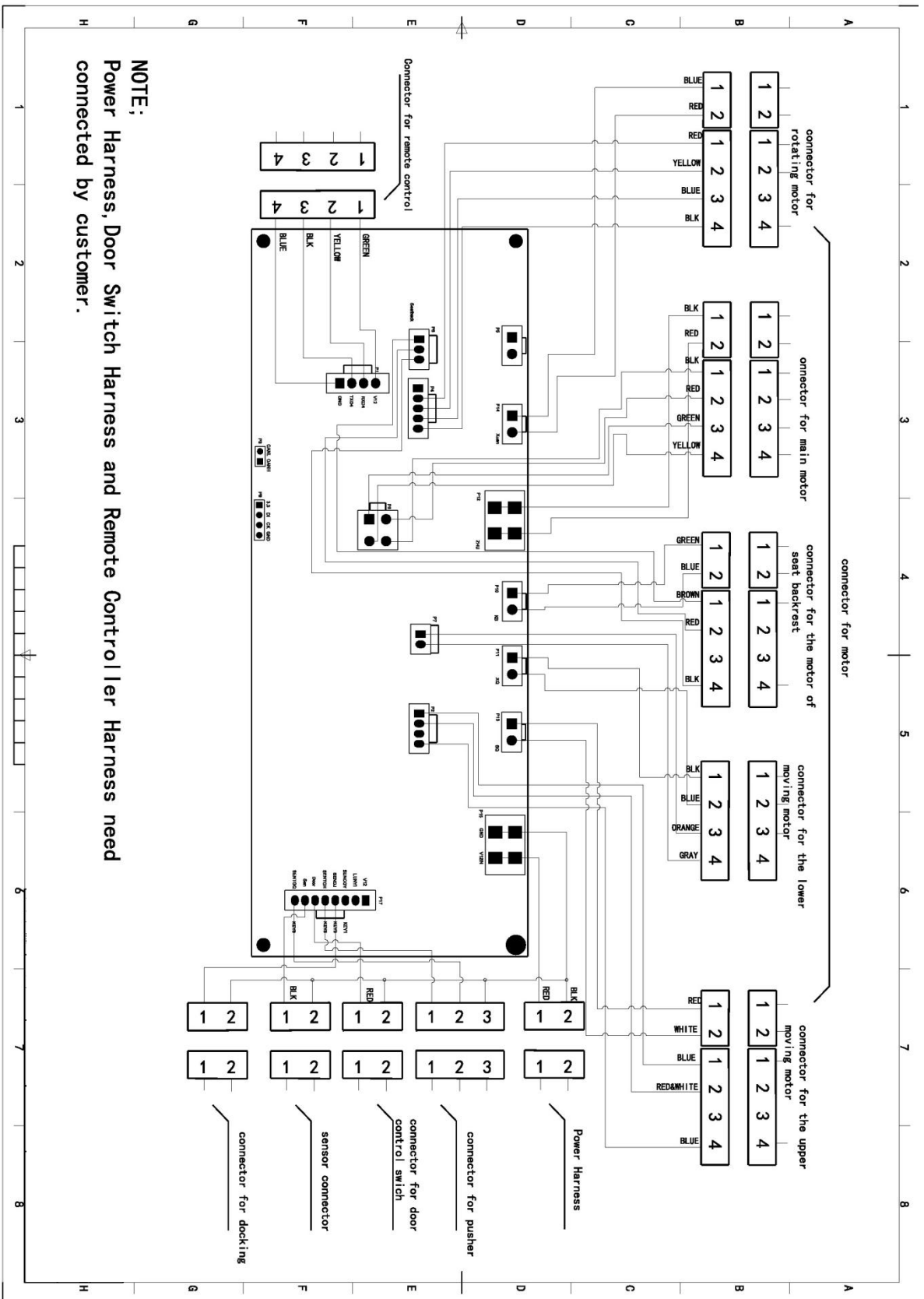
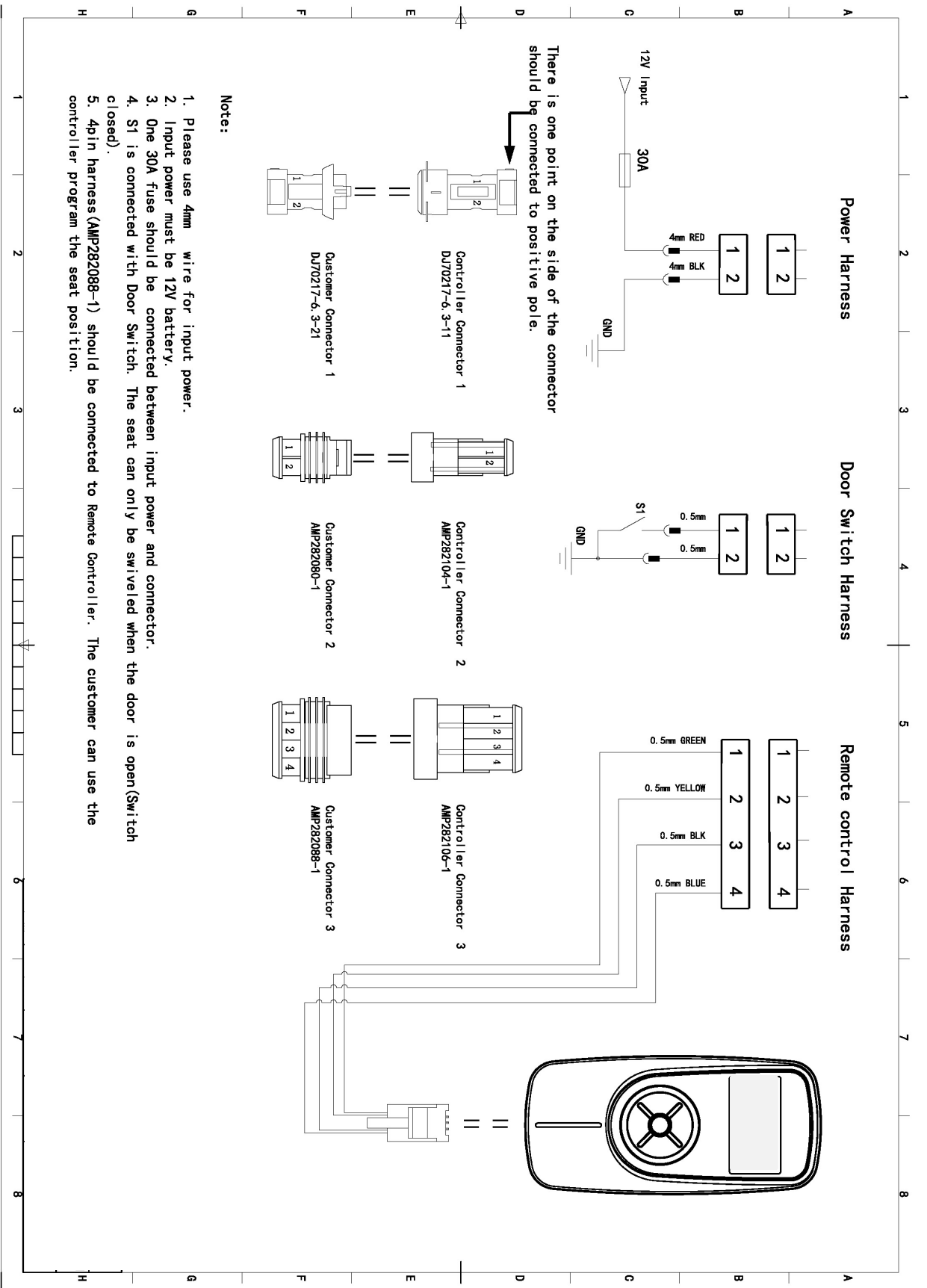
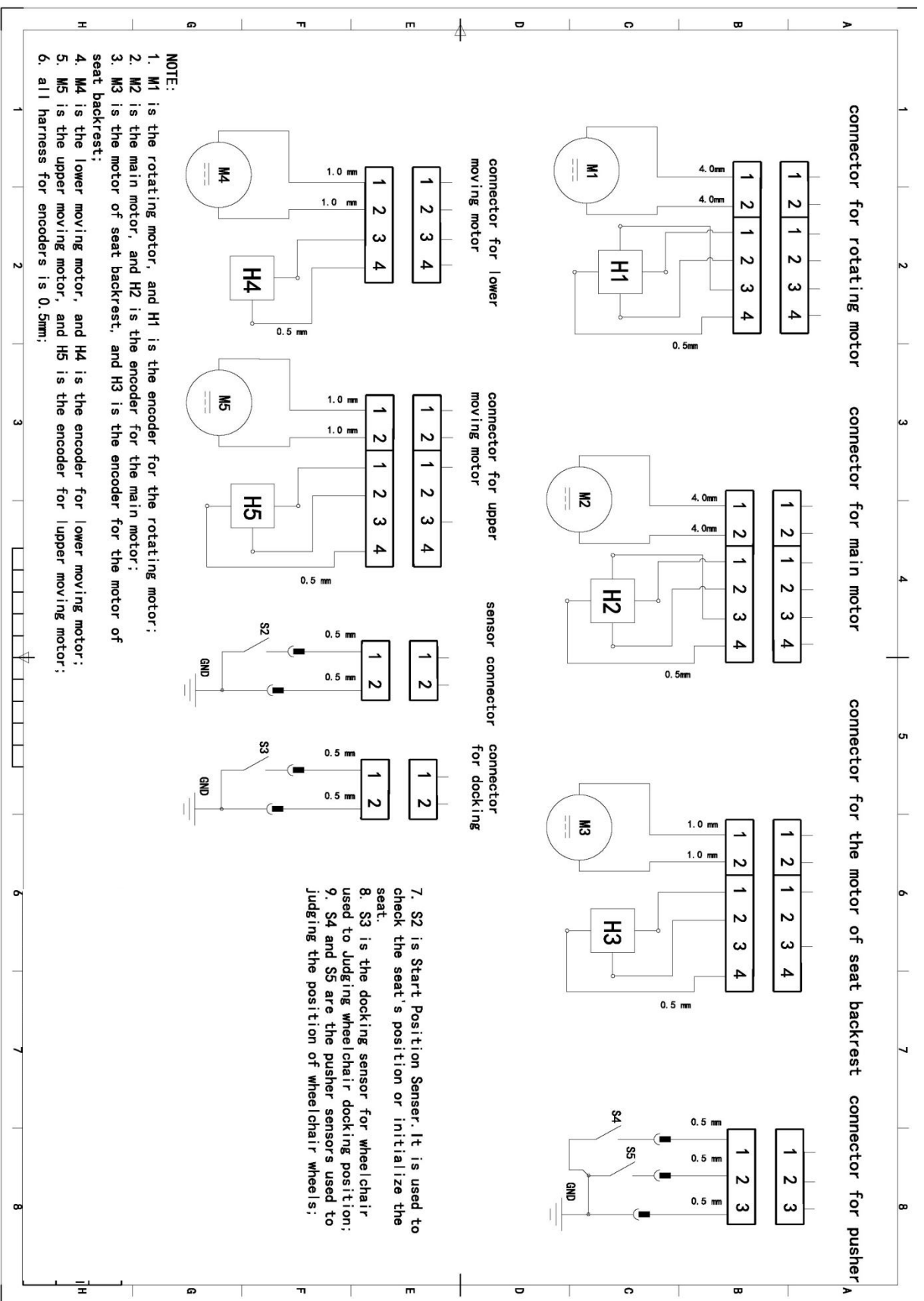
07/ 2023 - REVEN02

Table of Contents

1. Introduction.....	1
1.1. Technical	1
1.2. Dimension	2
1.3. Mounting Space Requirements	3
2. Standard Packing List	3
3. Precautions.....	4
4. Mechanism Installation.....	5
4.1 Parts List of Universal Mounting Bracket	5
4.2 Installation Steps	6
5. Electrical Installation.....	9
6. Install Mechanism Covers and Seat	10
6.1. Install bottom covers and motor cover	10
7. Normal Mode	11
7.1 Icons on Remote Control.....	11
7.2 Buttons on Remote Control	11
7.3 Switch Language.....	12
7.4 Automatic Rotate in and Rotate Out.....	12
7.5 Adjust Seat Position.....	13
7.6 Adjust Seat Backrest.....	13
7.7 Dock wheelchair to Mechanism	14
7.8 Switch to extreme cold mode	15
7.9 Manual Operation Mode.....	15
7.10 View Running Cycles.....	18
7.11Set the max stroke of moving forward or backward and set the max tilt angle of seat backrest.....	18
7.12 Backrest reset automatically	19
8. Programming Mode	20
8.1. Enter Programming Mode.....	20
8.2. Menu of Programming Mode.....	20
8.3. Programming Instruction	20
9. Transmission Mode.....	21
9.1. Enter Transmission Mode.....	21
9.2. Menu of Transmission Mode	22
9.3. Functions of Transmission Mode	22
10. Sleep Mode.....	22
11. APP Bluetooth mode and mobile APP	23
12. Manual operation for emergency.....	23
12.1 Rotate mechanism	23
12.2 Movement of Bottom Mechanism	24
12.3 Movement of Upper Mechanism	25
12.4 Lift and lower mechanism	25

13. Troubleshooting..... 26

- 13.1. Transmission Error26
- 13.2. Motor Current Protection26
- 13.3. Motor Short Circuit27
- 13.4. Motor Encoder Error27
- 13.5. Cannot extend out under programming mode28
- 13.6. Home Position Error28
- 13.7. Low Voltage.....28

Attachment 1: electrical wiring diagram 29

Preface

Thank you for choosing swivel seats from Guidosimplex. We hope our swivel seats will bring you unprecedented autonomy, comfort and relaxation in your daily life!

Our Products are exported to more than 50 countries and regions in the world.

This manual includes the after-sales declaration, product introduction, technical parameters, installation instructions, operation instructions, troubleshooting and maintenance instruction. In the process of your installation or use, please contact Guidosimplex , and we will provide you with a timely and satisfactory solution.

The version number of this manual is V2.0.

Guidosimplex reserves the right of alternations without previous notice.

Thank you again for choosing the products of Guidosimplex. Wish you a happy life!

GUIDOSIMPLEX srl - Via del Podere S. G iusto, 29 - 00166

Roma (Italy) Tel. +39 06.6152801 - Fax +39 06.61528036 -

Email infoclient@guidosimplex.it

www.guidosimplex.it

1. Introduction

STRAN Plus series swivel seat is a detachable welfare seat which installed in the middle row of MPV (applicable to commercial vehicles with sliding doors). Replacing the original car seat, this seat swivels out of the vehicle and lowers in height.

Compared with the STRAN Plus series swivel seat, the seat can be released from the mechanism, and the user does not need to transfer or move to other wheelchairs or equipment through the docking device. After the seat released, it can be used directly as a wheelchair, which can save the cumbersome operations and time waste caused by the secondary transfer.

STRAN Plus series swivel seat makes travel more accessible for the elderly and people with disabilities.

The released wheelchair has manual one and electric one for option.

This swivel seat can be operated by wired remote control or phone APP.

The turning direction can be clockwise or counterclockwise.


1.1. Technical

- Voltage: 12V dc;
- Power: 120W;
- Loading Capacity: 120Kg;
- Lowering Height (h): 390 mm;
- Rotating Angle: 0-90°;
- Quiescent current: ≤0.3mA
- Max current: 25A;


1.2. Dimension

Model	wheelchair	Gross length A	Gross height B	Gross width C	Rotating angle D	Floor height E	Height of wheelchair F	Install position
S Plus E	electric	800	1020	640	0-90°	≤560	500	Middle row
S Plus	manual	800	1020	640	0-90°	≤560	500	Middle row

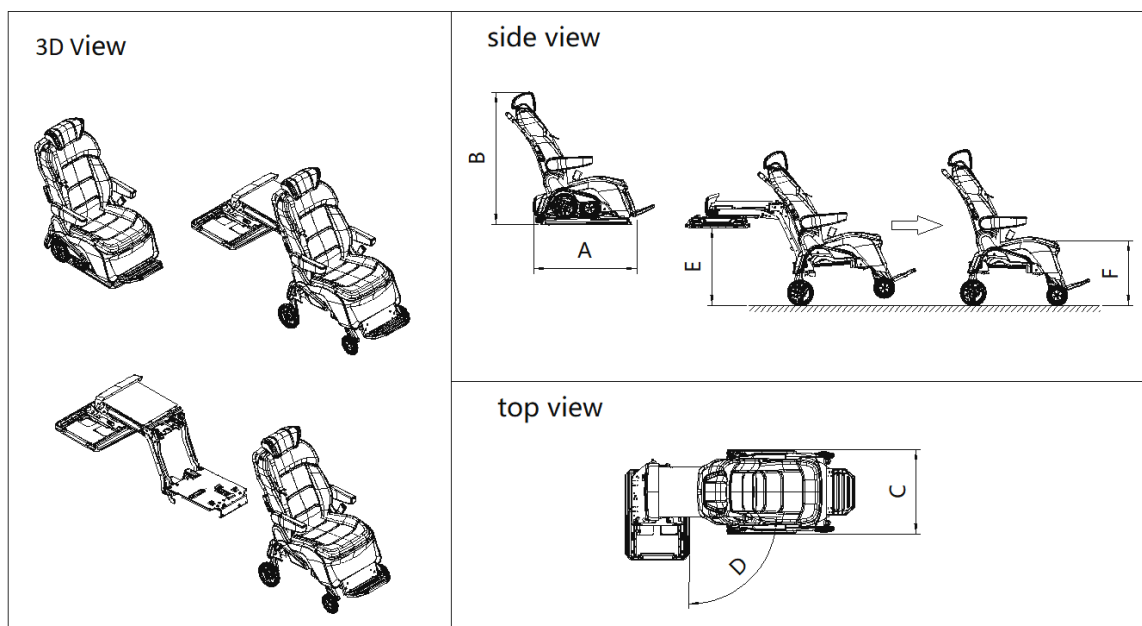 **Please do not modify the product, otherwise it will directly affect your after-sales service!**

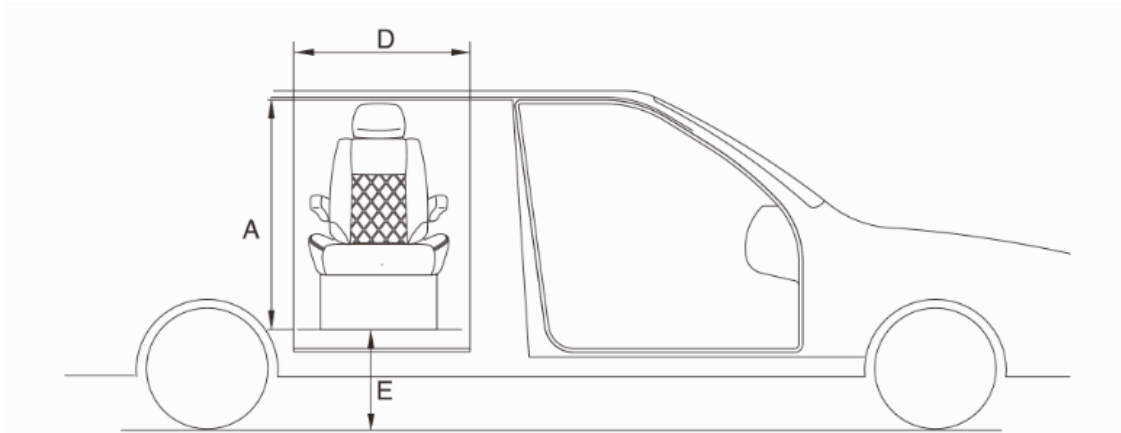1.3. Mounting Space Requirements

Model	Minimum installation space required			Installation position
	A	D	E	
SLIFT PLUS	≥970	≥660	≤540	Middle row

2. Standard Packing List

NO.	Name	Qty	Remark
1.	STRANPlus E / STRAN Plus swivel seat	1	
2.	Mounting brackets	1	option
3.	Mechanism cover	1	
4.	Manual tool kits	1	
5.	Electrical accessory kits	1	
6.	Packing list	1	
7.	List of manual tool kits and Electrical accessory kits	1	
8.	List of mechanism covers	1	

See the packing list in the box for details.

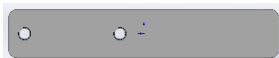
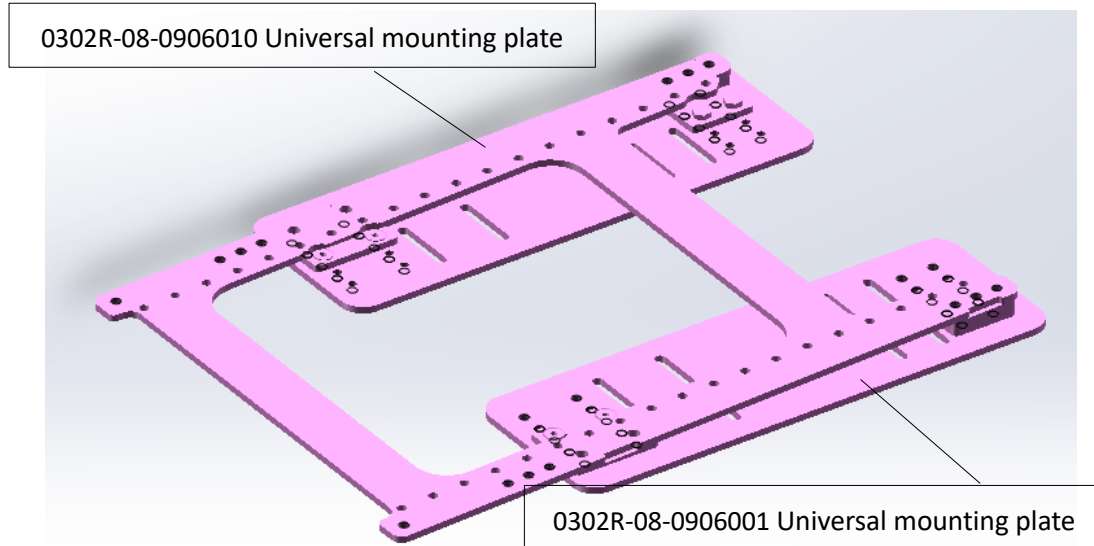
3. Precautions

WARNING: The weight of seat mechanism is around 80KG. Please do not install it alone. There should be at least 2 people installing this seat together to avoid accidental injuries.

- Check if there is any missing part according to the standard packing list in chapter two. We always put one packing list in the mechanism package.
- Please wear protective clothing and eye protection in case of cutting or welding.
- Check if there is enough mounting space.
- Read and thoroughly understand this installation manual before installation.
- Please install this seat carefully to avoid injuries. Be sure that your hands, feet, legs, and clothes are not on the product movement trajectory.
- Don't sit on this seat or put things on it before installation finished.
- After installation, check the operating current in the empty state of the mechanism, if the current value exceeds 7A, you need to check if there are obstacles to the normal movement of the product.
- Test it with rated capacity of 125% after installation.
- Check to see if this seat is installed well. Please don't use it if there is any unsafe and unusual noise or movement.
- Do not stand on the product movement trajectory.
- This installation manual is for model STRAN-R Plus which turning clockwise. Description is opposite if your seat model is STRAN-L Plus which turning anti-clockwise.
- This installation manual is based on universal brackets. If your seat mounting bracket is customized, please contact us if you face problem during installation.

4. Mechanism Installation

4.1 Parts List of Universal Mounting Bracket

0302R-08-0906002 Universal slider block(rear)


0302R-08-0906003 Universal slider block(front)


NO	Part code	Part name	QTY	Remark
1	0302R-08-0906010	Universal mounting plate	1	
2	0302R-08-0906001	Universal mounting plate	2	
3	0302R-08-0906002	Universal slider block(rear)	2	
4	0302R-08-0906003	Universal slider block(front)	2	
5	M8×12	Hex head bolts fully threaded (grade 12.9, black)	4	6 units as spare part
6	M8×35		14	
7	M8×12	Hexagon socket head screw (grade 12.9, black)	4	
8	M8×16		14	
9	M8×35		10	spare part
10	0516-386	Thread glue (Medium Strength)	1	

4.2 Installation Steps

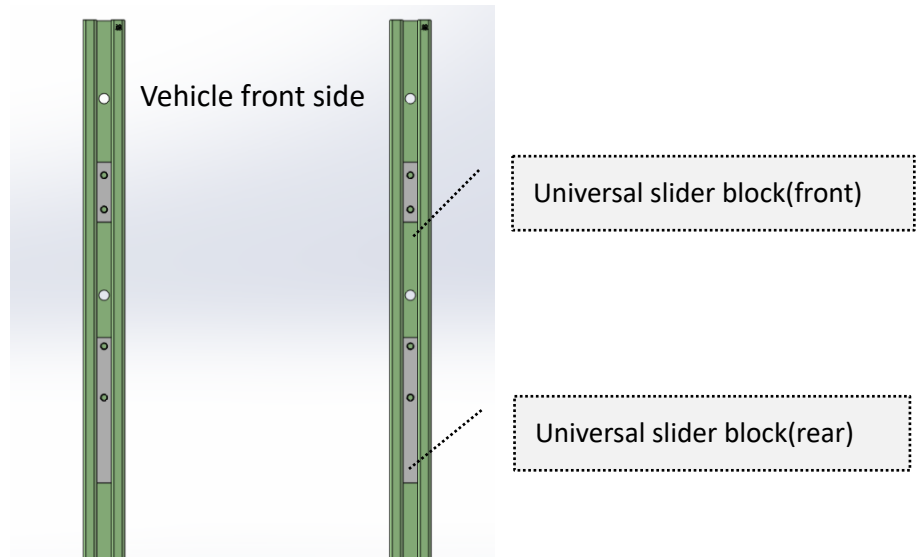
- **STEP1:** put the two kinds of slider blocks inside of vehicle seat rails. Please pay attention that the slider blocks are different.

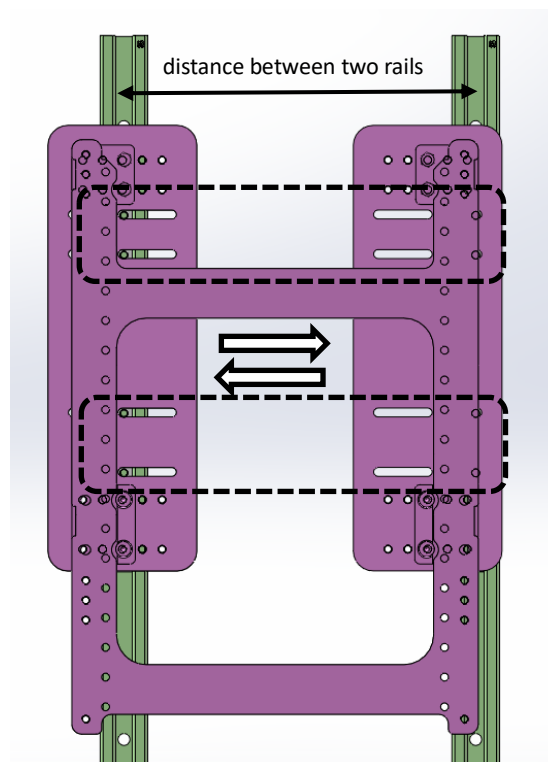
- **STEP2:** put the universal mounting plates on seat rails, and move right and left to see the fixing holes of slider blocks. If the distance between two rails is over 305mm, it may cause the slider blocks on the other side are not visible. In that case, you can first install the holes on one side, and install the holes on the other side by adjusting the universal mounting plate.

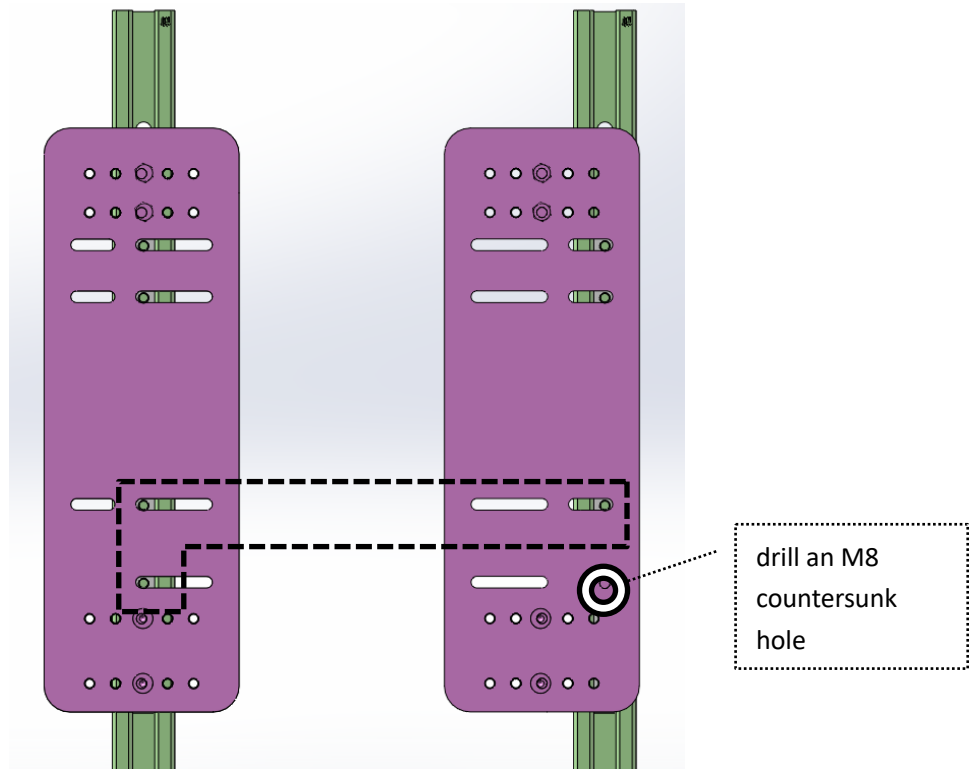
- **STEP3:** Adjust the position of universal mounting plates; (If installation of STEP 2 can be realized normally, this step can be ignored) Remove the upper mounting plate, and use bolts to fix in the visible holes. A hole in the lower right corner needs to be drilled with an M8 countersunk hole because hexagon head bolts cannot be installed;

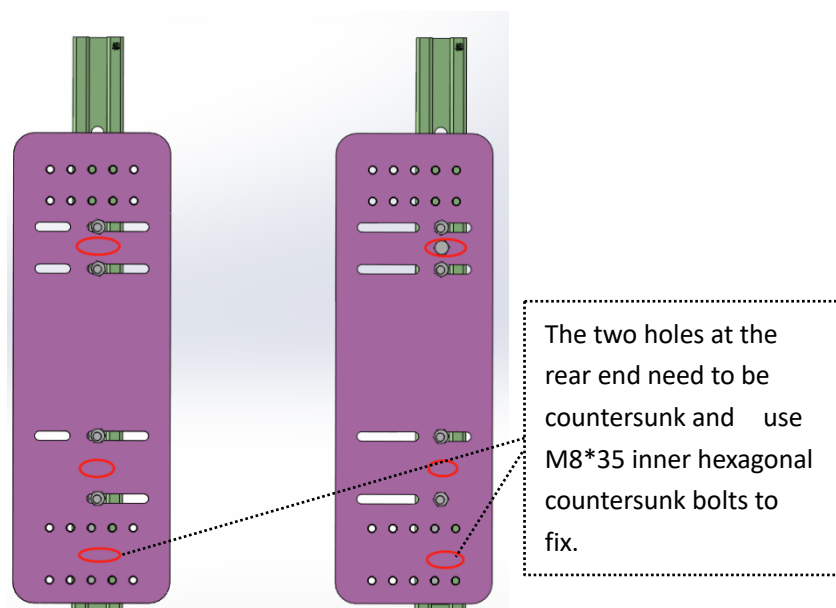- ✓ **STEP4:** Reinforce the slider blocks in the rails to prevent vehicle from shifting when turning. At the position of the red circle, according to the actual situation, drill holes that meet $\phi 7$, then tap M8 threaded holes on the slider blocks in the rails. Ream holes on the base mounting plate to $\phi 8.5$, and use M8*35 hex head bolts to fix;


- **STEP5:** Mount the mounting plate; If you don't do the installation in STEP 3, the mounting plate is not disassembled, please ignore this step;
- **STEP6:** Power on the mechanism of swivel seat, enter the manual mode (refer to 7.9 manual mode operation), rotate the upper mechanism 90 ° and then extend completely out until the for wheels unfolded, lift the wheelchair upwards and move it away, and then the mechanism can be installed and connected with 14 pieces of M8 threads on the two sides of the mounting plate.

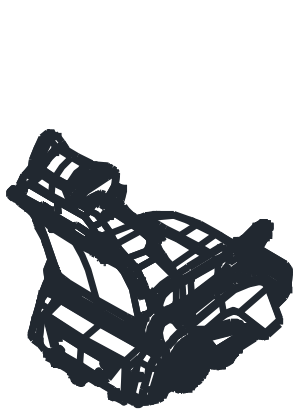initial state

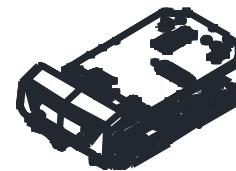
extend completely out until
the for wheels unfolded

mechanism without
wheelchair

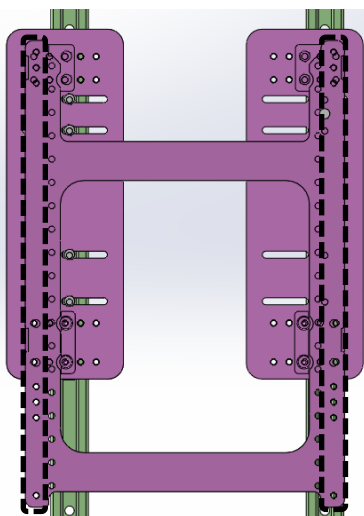- **STEP7:** Lift the wheelchair onto the mechanism, enter the programming mode and check whether the wheelchair can enter and exit the vehicle smoothly, and at the same time adjust the position of mounting plate to find the most comfortable mounting position;


When installing the 14 bolts on the bottom plate, first screw on the bolts but do not tighten them, test the forward and backward movement of the seat in manual mode to make

sure it can work normally. Then tighten the bolts, and test the forward and backward movement of the seat again after tightening all the bolts to ensure normal operation. If the current limiting protection occurs, do not continue to operate it, otherwise the motor parts may be damaged. Please check and loosen the bolts, re-adjust the installation surface of the bottom plate, and ensure that it is not deformed or twisted after installation. Repeat this until the mechanism can work normally and the current limiting protection is eliminated.


During the final installation, use thread glue to tighten the bolts to prevent the bolts from loosening.

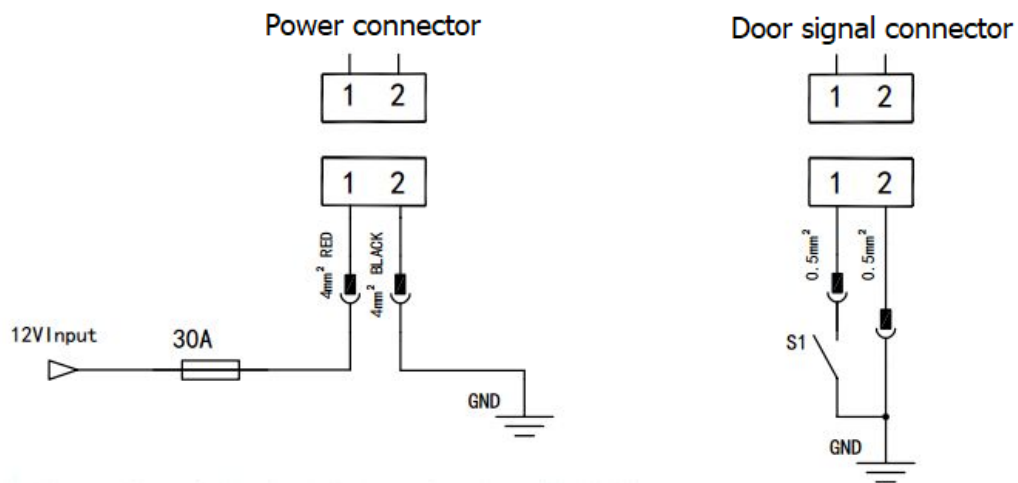5. Electrical Installation

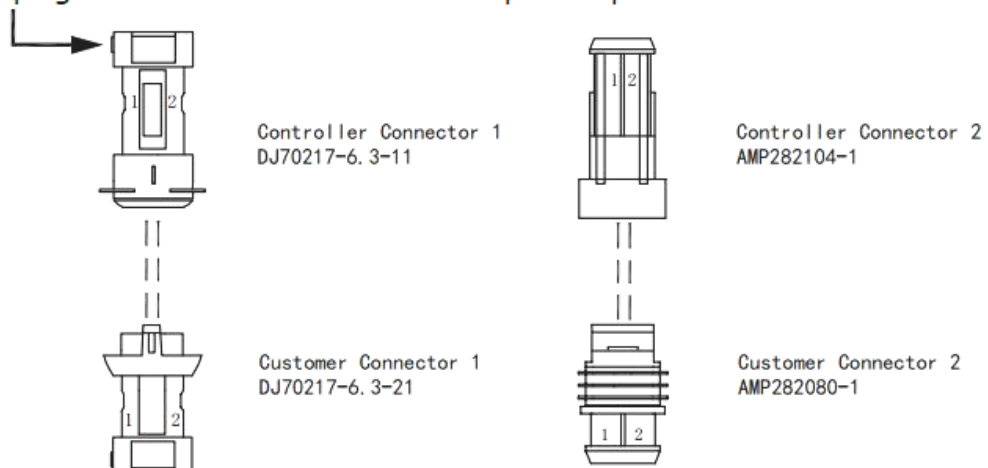connect plug of connector DJ70217-6.3-11 to positive power connector

Refer to above figures to connect the connectors.


DJ70217-6.3 is power connector. Connect red wire to power 12V and connect black wire to ground. AMP282080 is door signal connector. Connect black wire to ground and connect brown wire to door switch.

6. Install Mechanism Covers and Seat

After mechanism installation finished, rotate it out of vehicle by manual mode, and then you can install the mechanism covers. Refer to the operation of manual mode.

6.1. Install bottom covers and motor cover

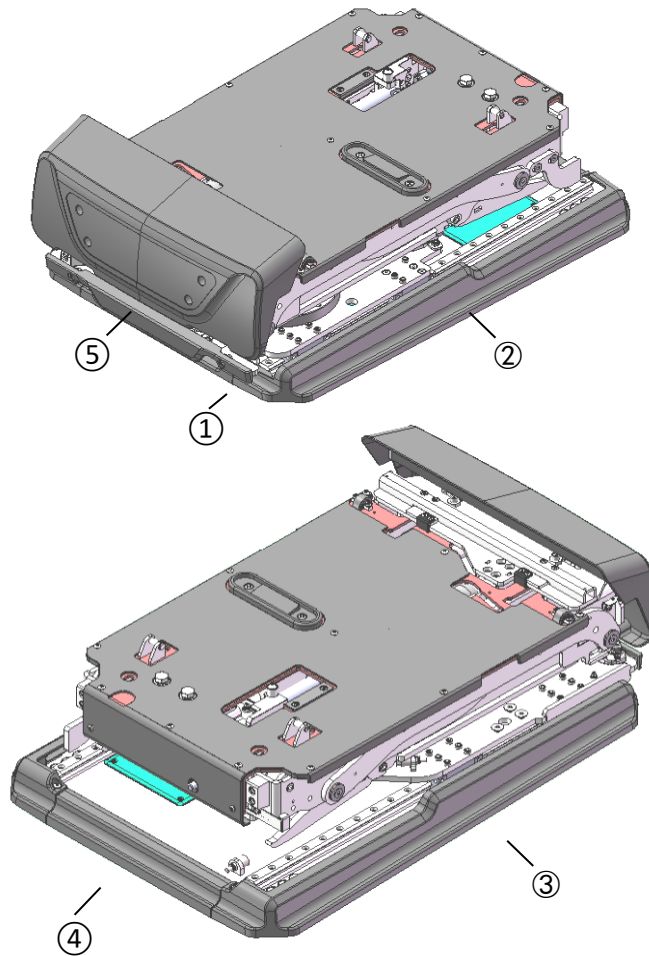Refer to below figures. First install the bottom covers and motor cover.

Parts list:

NO.	Reference code	Part name	QTY
1	0302R-09-0000014	Bottom cover (rear)	1
2	0302R-09-0000016	Bottom cover (right)	1
3	0302R-09-0000015	Bottom cover (left)	1
4	0302R-09-0000017	Bottom cover (front)	1
5	0302R-09-0000008	Middle plate motor cover	1
6	0516-678	Cross groove large head screw M4*8	4

Please refer to cover packing list to see details which you can find it in the parts packing box.

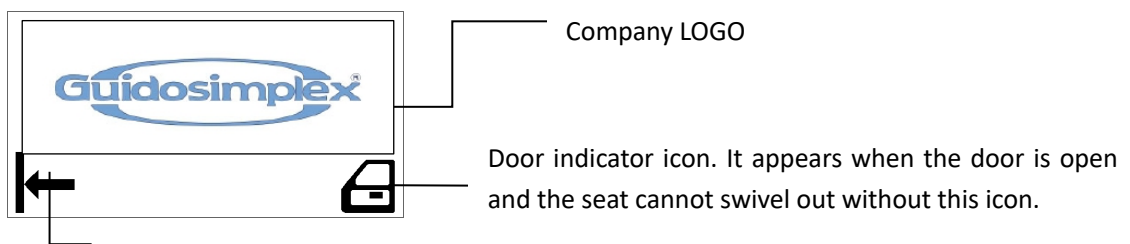
7. Normal Mode

under normal mode, this seat will work according to the programmed path to rotate in, rotate out and adjust backrest.



Don't swivel out your seat when you park your car on slope, your seat may not work properly.

7.1 Icons on Remote Control

Icon of Programming position. When this icon appears, it allows you to program path.

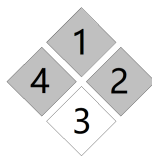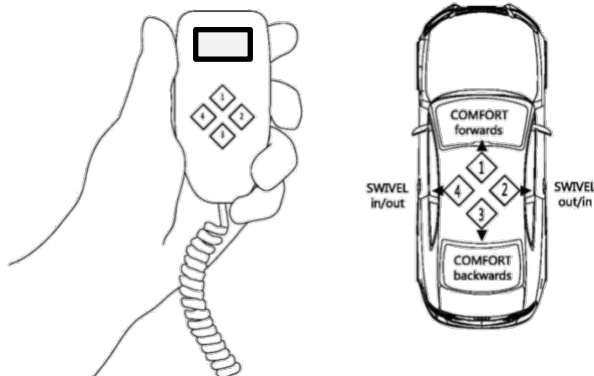7.2 Buttons on Remote Control


When door open:

Button 2: swivel seat out;

Button 4: swivel seat in;

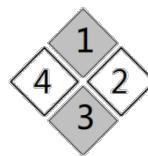Button 1: swivel seat in (when there is no wheelchair on mechanism);

When door close:

Button 1: move seat forward;

Button 2: not available for operation;

Button 3: move seat backward;

Button 4: rotate seat in;

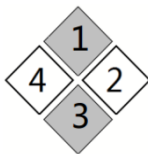
WARNING: This seat will stop work immediately when you release buttons.


Button indicator light will be on if it is available for operation, otherwise it is no function.

This user manual is for model STRAN-R Plus which turns clockwise. If your seat model is STRAN-L Plus, the function of button 2 and 4 is opposite.


7.3 Switch Language

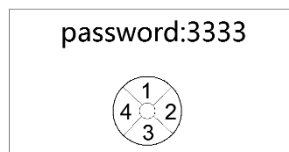Press button 1 and 3 simultaneously for 5 seconds



Enter password: 3333

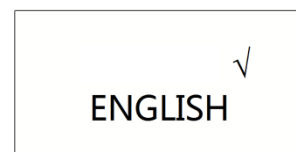
Press button 1 or 3 to choose language; Press button 2 or 4 to confirm selection.

7.4 Automatic Rotate in and Rotate Out

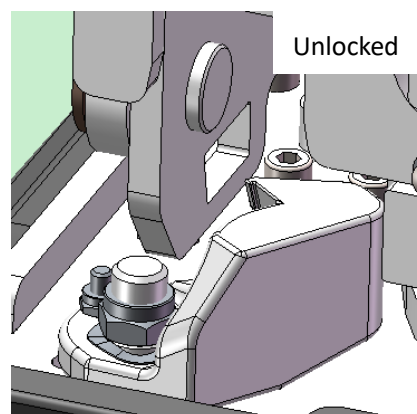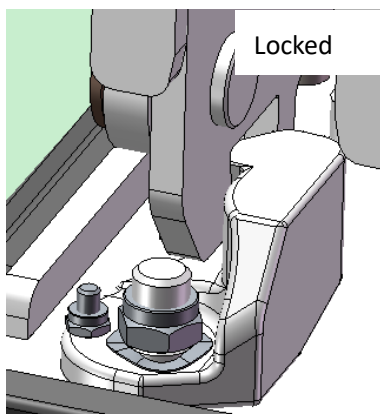- When the seat starts to swivel out, the seat will rotate out automatically according to the programmed path. At this time, the function of moving seat forward and backward and adjusting backrest is not available.
- Before the seat rotating out, it will first unlock itself. It will move to the rearmost position and move forward a little bit to unlock itself. If this seat is unlocked but the door is closed, it will automatically rotate in and move forward to lock itself.

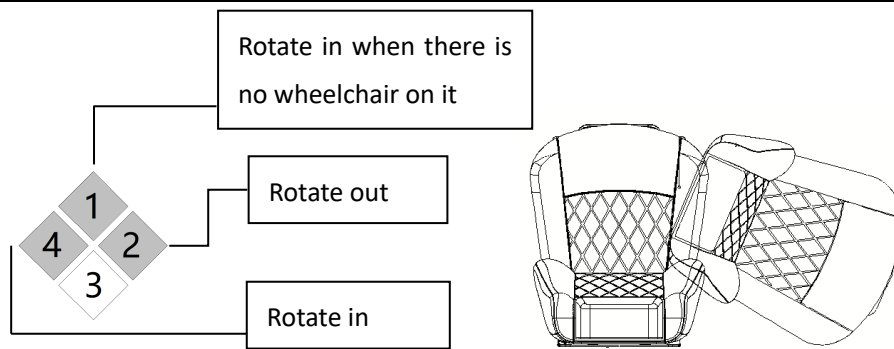- When the seat starts to swivel in, it will rotate in automatically according to the programmed path. After this seat fully rotates in, it will automatically return to the position that the last procedure performed (i.e. automatically return to the user's "comfort position").


This function is only available when the door is open.

7.5 Adjust Seat Position

- This function is not available when seat starts the procedure of rotating;
- You can freely move this seat forward or backward after it rotates in completely;

Button 1: move the seat forward;

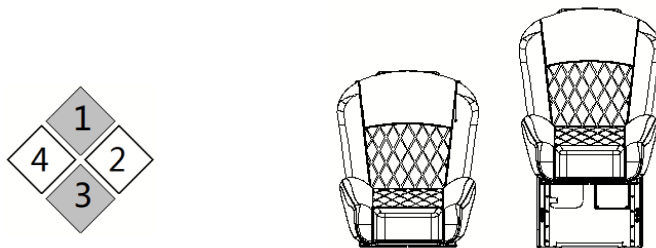Button 3: move the seat backward;


7.6 Adjust Seat Backrest

This function is not available when seat starts the procedure of rotating.


You only can adjust seat backrest after seat rotates in completely.

Button 2: Tilt backrest forward;

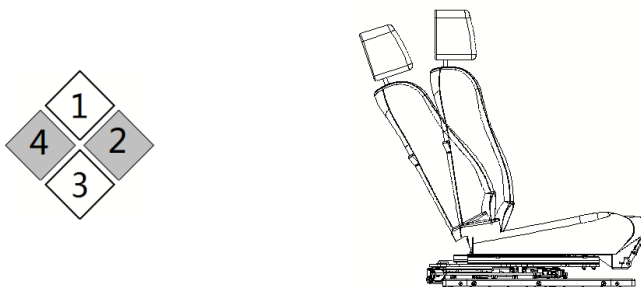Button 4: Tilt backrest backward;

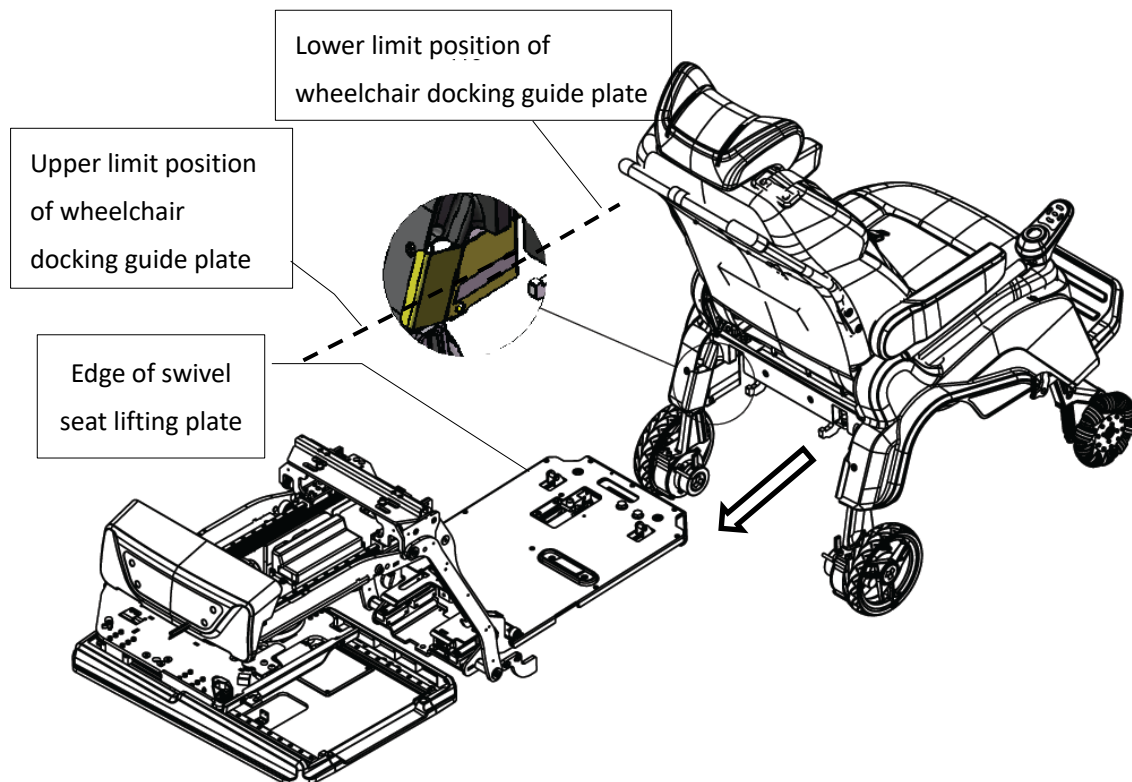7.7 Dock wheelchair to Mechanism

- As shown in the figure above, when the wheelchair is docked with the swivel seat mechanism, it is necessary to make the upper edge of the lifting platform as flush as possible with the lower limit position of the wheelchair docking guide plate, so as to minimize the docking difficulty caused by the uneven road surface.
- When docking electric wheelchair, the wheelchair driver can reverse the wheelchair and dock with the mechanism. After hearing the "beep" sound, it means the docking is done successfully, and then retract the wheelchair into the car by remote control.
- Meanwhile, this step can also be assisted by the accompanying staff. He/she can press and hold the button "+" and "-" on the control handle of the wheelchair at the same time. After hearing the sound of "beep", it means that the wheel has been unlocked. The accompanying staff can manually push the wheelchair to dock with the mechanism. Please hold the wheelchair during docking to avoid the uneven road surface causing the wheelchair to slip and affect the docking.
- If the wheelchair you bought is a manual one, when docking it, the accompanying staff should manually push the wheelchair to dock with the mechanism. When docking, pay attention to holding the wheelchair or directly pulling up the brake of the wheelchair after hearing the "beep" sound. At last, retract the wheelchair into the car by remote control.

7.8 Switch to extreme cold mode

In case the vehicle is parked for a long time in extreme cold weather, when use this swivel seat for the first time, it may increase the resistance of the mechanism. At this time, you can warm up the vehicle for 3-5 minutes and then use this swivel seat, or you can enter the extreme cold mode, and then use this swivel seat.

Operation:

Long press button 1 and button 3, and enter password "1212" to enter the extreme cold mode; there is a icon ❄️ on remote control. Other operations are same with the normal mode.

After this swivel seat returns to the car completely and the door is closed, the swivel seat will automatically switch to the normal mode.

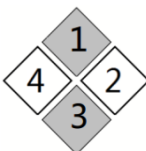
7.9 Manual Operation Mode

You can switch to manual operation mode by the remote control.


The main two functions of manual mode:

- Under emergency situation, you can operate this seat to fulfill every movement by the remote control. At this time, this seat will not work according to the program.
- When the program is in disorder, you can use manual mode to return this seat to programming position and reset it.
- If you want to reprogram your seat, you have to switch to manual mode and return the seat to programming position.


7.9.1 How to switch to manual mode

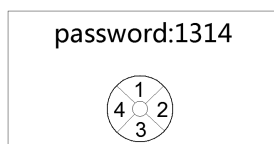Press buttons 1 and 3
simultaneously for 5


Enter password:
1314

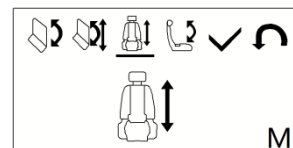
Main interface of
manual mode

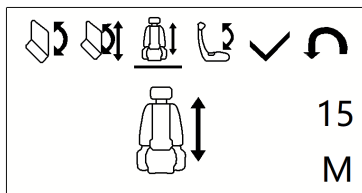7.9.2 Menu of Manual Mode

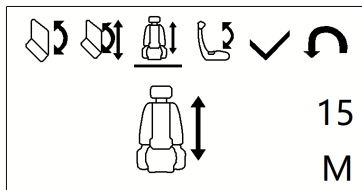- 7 menu interfaces. use button 1 and 3 to switch interface you need
- This number represents the operating current.
- “M” means manual mode. It will not show under


Rotate seat out/retract seat in

Button 2: rotate seat out;

Button 4: retract seat in;

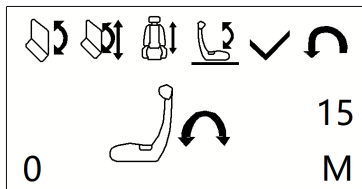Under programming mode, the seat rotation angle should be more than 45°, otherwise this function is not available.


Adjust backrest

Button 2: tilt backward;

Button 4: tilt forward;

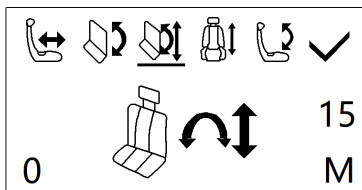“0”: this number means angle of seat backrest. If it is 0, backrest is vertical.


Rotate seat and move seat forward/backward at same time

Button 2: rotate seat out and move forward; If seat is still locked, the interface displays blocking information;

Key 4: rotate seat in and move backward;

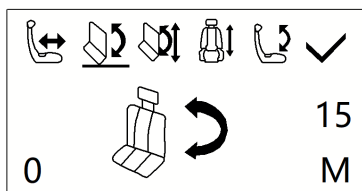“0”: means the rotating angle;


Rotate seat

Button 2: rotate seat out; If seat is still locked, the interface displays blocking information;

Button 4: rotate seat in;

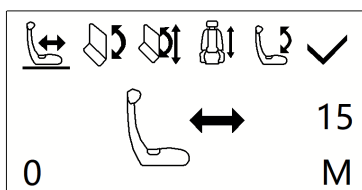“0”: means the rotating angle;


Move seat forward/backward

Button 2: move seat forward;

Button 4: move seat backward;

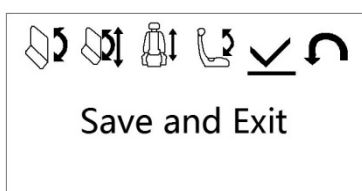
“0”: means seat position; if it is 0, the seat is at programming position;

Save and exit

Button 2 or 4: save and exit;

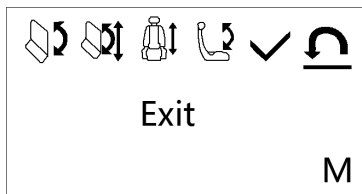The path will be saved under programming mode;

Exit directly

Button 2 or 4: exit directly;

The path will not be saved under programming mode;

7.9.3 Manual mode under emergency situation

If your seat has been rotated out, but it cannot rotate in, you can switch to manual mode to rotate it in.

Please refer to chapter 7.9.2 to know how to use manual mode.

Please note that when you use manual mode to operate your seat, it will not work as the programmed path. The better way is using manual mode to return your seat to programming position and reset it.

If the motor encoder is OK, you can exit manual mode directly without resetting, the program will lead the seat back to the programmed path by itself.

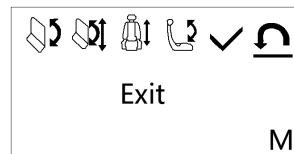
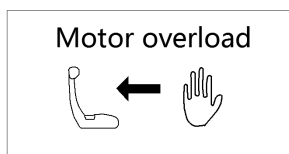
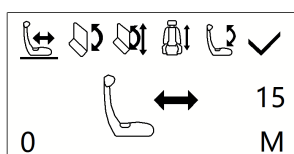
7.9.4 Reset seat by using manual mode

You need to reset the seat by using manual mode under the situations below:

- Reprogram your seat;
- Motor replacement;
- Program is in disorder because of motor encoder failure;



Follow the steps below to reset your seat by using manual mode:

- First, please make sure your seat has been rotated in completely. (refer to chapter 7.9.2 to rotate seat in manually).
- Second, move seat backward until the interface shows motor over current protection (refer to chapter 13.2).
- Finally, exit manual mode directly. At this time all indicator lights of remote control are off and backrest will tilt forward to reset the seat program.
- If the programming position shows  , this seat has been reset successfully.

7.10 View Running Cycles

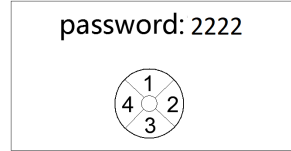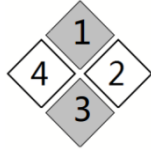
Press buttons 1 and 3 simultaneously for 5 seconds, and then Enter password: 2222

7.11 Set the max stroke of moving forward or backward and set the max tilt angle of seat backrest

The movement of backrest will be memorized only in the first step of programming (you can adjust the backrest before mechanism turning to meet the tight space) , and the other movements of backrest will not be memorized, so you have to set separately through this menu.

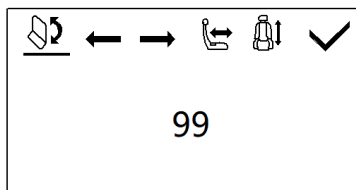
Operation method: long press button "1" and "3", enter the password "1313" to enter the parameter setting interface, and use button "1" or "3" to switch the interface.

Set the rotation angle of the backrest when it starts to tilt backward

Use button "2" or "4" to adjust the value;

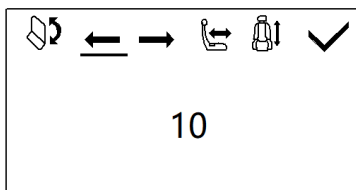
The larger the value, the later the seat backrest starts to tilt backward;

Set the angle of tilt backward before leaving the car door

Use button "2" or "4" to adjust the value;

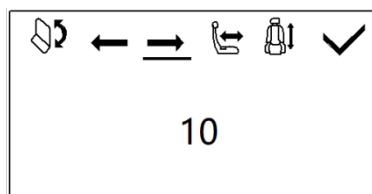
The larger the value, the larger the angle of tilt backward before leaving the car door;

Set the angle of tilt forward after leaving the car door


Use button "2" or "4" to adjust the value;

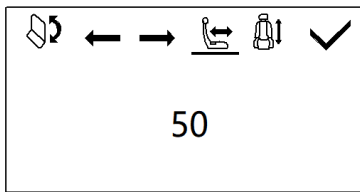The smaller the value is, the more vertical the seat backrest is after leaving the car door;

Set the maximum distance the seat can move forward in the car

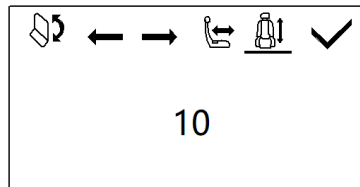Use button "2" or "4" to adjust the value;

Set the distance from the door when the seat backrest starts to tilt forward;

Use button "2" or "4" to adjust the value;

The larger the value, the later the backrest starts to tilt forward;

After the setting is finished, press button 1 or 3 to switch to the ✓ confirmation interface, and press button 2 or 4 to confirm the setting.

7.12 Backrest reset automatically

After this seat works for 100 cycles, the seat backrest will reset automatically.

Reset process:

When seat return into car completely, continue to press button 2, the seat backrest will tilt vertically and then recover to comfort position. If you release button 2 during the reset, this seat will reset the backrest when return into car next time.

Please do not stop the seat backrest intentionally while resetting, otherwise it may be out of position.

8. Programming Mode

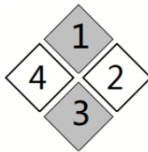
You can program your seat by using the remote control.

If the electric seat is used, the movement of the seat backrest will be completely synchronized with other movements of the mechanism, and the movement of seat backrest can be set by the remote control. Meanwhile, you can set the max stroke of moving forward in the car through the remote control which can avoid the risk of collision. Refer to chapter 7.9 to set it.


It is recommended to backup the original procedure by using the remote control first, and then reprogram your seat in case that you are not satisfied with the new procedure. Refer to chapter 9 to know how to backup and restore the procedure.

8.1. Enter Programming Mode


First reset your seat by using manual mode and make sure the programming icon is displayed on left bottom of screen. Refer to chapter 7.9 to know how to reset seat by manual Mode.

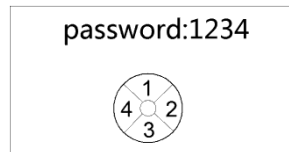Press button 1 and 3
simultaneously for 5 seconds

Enter password:
1234

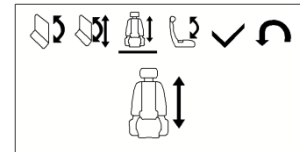Menu of programming
mode

8.2. Menu of Programming Mode

The menu of programming mode are almost same with menu of manual mode. Difference is that the remote control will not display icon "M" under programming mode. Refer to chapter 7.9 to know the functions and operations of each menu.

8.3. Programming Instruction

You can operate the seat under programming mode, and the seat will save records automatically except the meaningless adjustments. Refer to chapter 7.9.2 to know how to operate seat under programming mode.

When seat receives the first programming instruction, it will unlock itself (rotate a little).

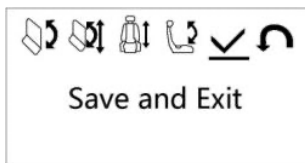
Generally, the programming starts from adjusting seat backrest.

Please note that if you program the seat to rotate less than 45°, it cannot extend out. It will remind you extending forbidden.

After programming finished, switch to menu of save and exit, and then press button 2 or 4 to same the procedure. At this time, the programmed procedure is saved. When you rotate seat in, it will rotated in automatically according to this procedure.

After programming finished, if you switch to menu of exit, it will not save the programmed procedure. Under this condition, please return the seat to programming position to reset it by using manual mode (refer to chapter 7.9.4). At this time, the previous procedure is restored.


9. Transmission Mode

The programmed procedure is save in the seat controller, so the procedure will not change even if you use another remote control.


But you can backup the procedure in remote control which is a big advantage. You can copy the procedure to cover the procedure which saved in the controller by using the transmission mode. Or you can extract the procedure in controller and backup to the remote control, and then copy this procedure to other seats in batch.

9.1. Enter Transmission Mode

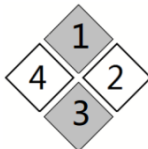
To enter the transmission mode, first reset your seat by using manual mode. Make sure the icon of programming position is indicated on the bottom left of remote control screen. Please refer to chapter 7.9.4 to know how to correct seat.

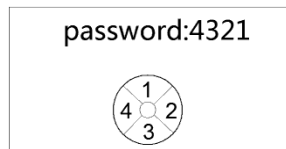Press buttons 1 and 3
simultaneously for 5


Enter password:
4321

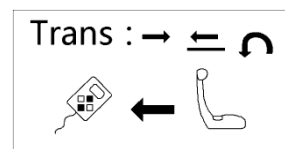Menu of
Transmission Mode


9.2. Menu of Transmission Mode

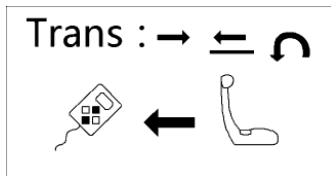There are three menus of transmission mode. Press button 1 and 3 to switch menu and press button 2 and 4 to confirm selection.

Backup procedure to remote control

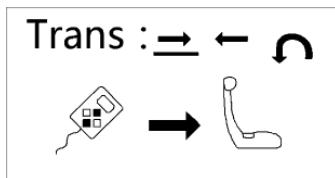This operation will copy the procedure to the remote control.

Cover the procedure in seat controller

This operation will copy and cover the procedure from remote control to seat controller.

This function is very useful for batch programming.

9.3. Functions of Transmission Mode

- **Procedure back up:** when you want to reprogram your seat, you can first backup the procedure to remote control. In case that you are not satisfied with the new procedure, you can copy the previous procedure from remote control to seat controller.
- **Copy procedure in batch:** when you order our seats in batch and want these seats to work in same procedure, you can use remote control to copy and cover procedures to seat control, in that case, all seats will work according to same procedure.

10. Sleep Mode

- Icon for door: 
- When door close, the door icon  will disappear.
- When the door close and the seat hasn't been operated for 10 seconds, the seat will enter sleep mode with low quiescent current.
- The seat will wake up immediately by pressing any button of remote control.

11. Bluetooth mode and mobile APP

Our seat has Bluetooth application, and can be operated by your cellphone.

To use the APP function, first connect your cellphone to the Bluetooth in our seat, and then set the Bluetooth mode.

Please refer to APP user manual to get more information

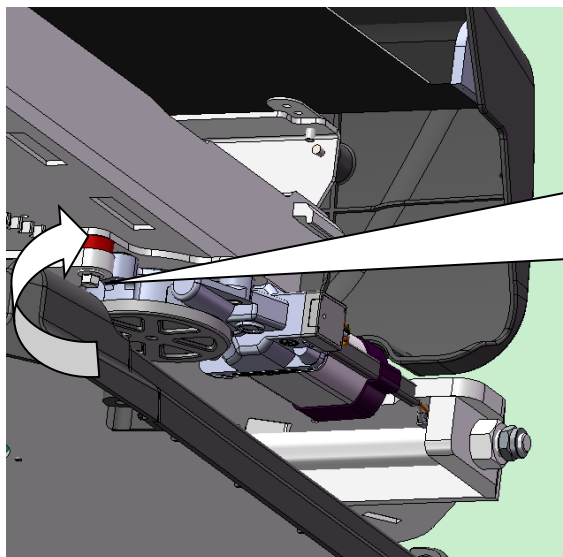
12. Manual operation for emergency

This seat can be rotated in manually if there is emergency situation. In that case, after rotating seat in, please drive your car to the repair shop to inspect and repair it to avoid serious accidents.


Follow below steps to rotate your seat in.

12.1 Rotate mechanism


Remove this bolt by an open spanner 7mm, and then rotate the motor. After the gear disengaged, this seat can be rotated in manually.

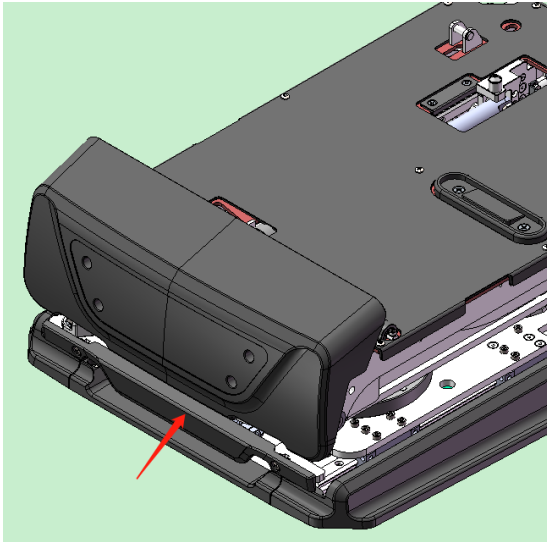
12.2 Movement of Bottom Mechanism

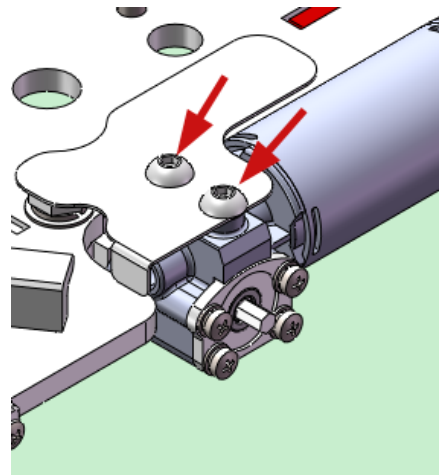
Remove motor cover which is at the rear of mechanism


Remove the cover plate of motor harness

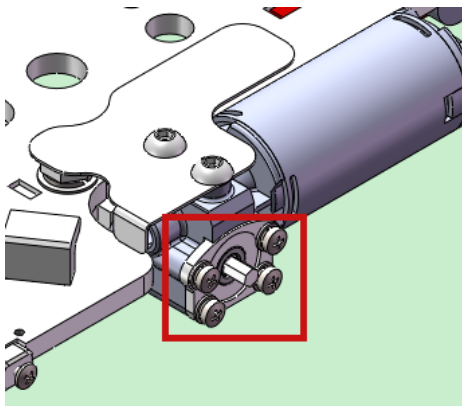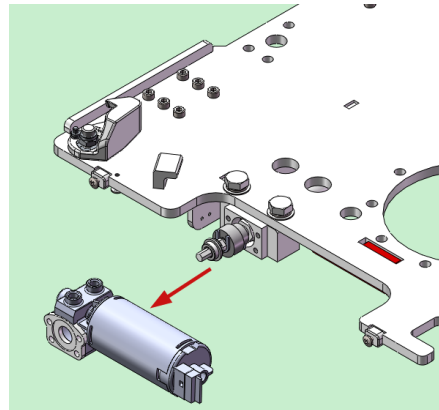Remove the 4 bolts which used to fix motor



Remove the motor

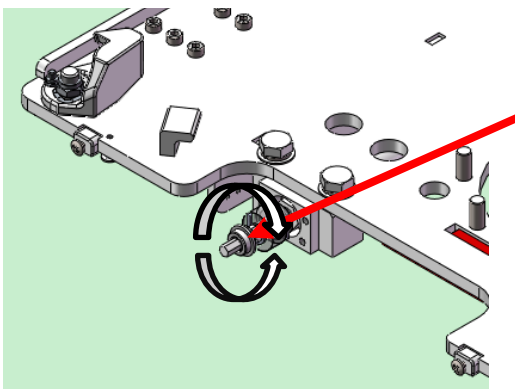Use a socket spanner SD6 to rotate the motor rotating shaft.
 Rotate it clockwise, the mechanism will move forward;
 Rotate it counterclockwise, the mechanism will move backward;

12.3 Movement of Upper Mechanism

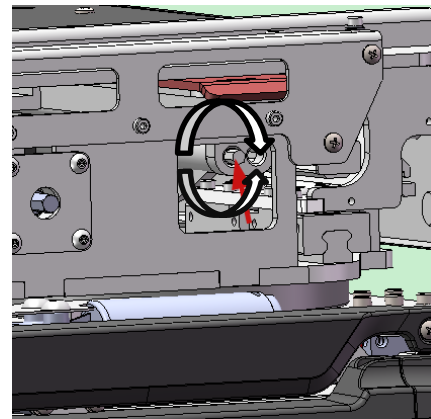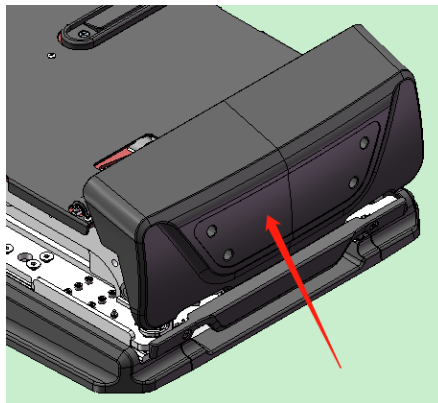1. Remove the rear cover of mechanism;
2. Use a SD6 socket wrench to insert the rotating shaft at the end of the upper motor;
3. Rotate the shaft clockwise, mechanism will move forward;
4. Rotate the shaft counterclockwise, mechanism will move backward;


12.4 Lift and lower mechanism

Use a SD8 socket spanner to insert the rotating shaft at the end of lifting motor.

Rotate it counterclockwise, the mechanism will move forward and lower;

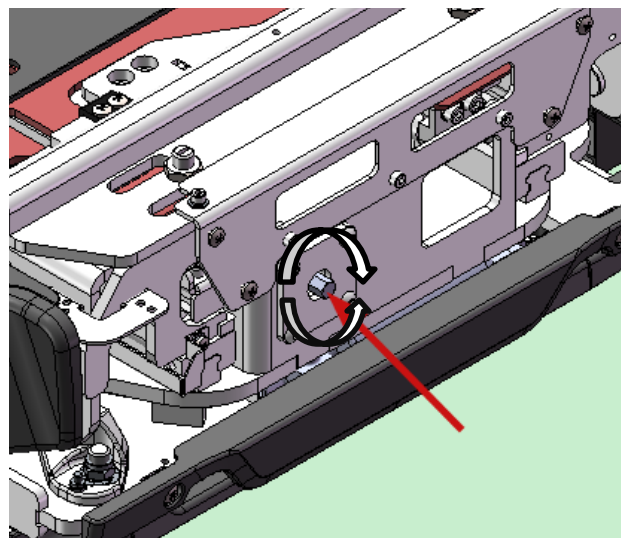Rotate it clockwise, the mechanism will lift and move backward;


13. Troubleshooting

When there is some failure, the fault information will be displayed on the remote control. You can repair it according to the troubleshooting.

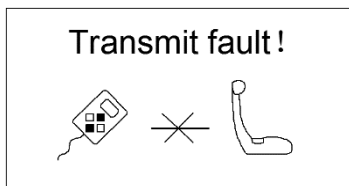13.1. Transmission Error

This icon indicates that communication between your seat and the remote control is lost.

Possible reason: The controller and the remote control are disconnected. Controller is failed.

How to repair: Check if there is any damage on the harness or try to replace controller.

After repaired, this icon will disappear.


13.2. Motor Current Protection

This icon indicates that motor running current exceeds limit value.


Remote control will display the running motor and it's moving direction.

Icons of current protection for different motors:

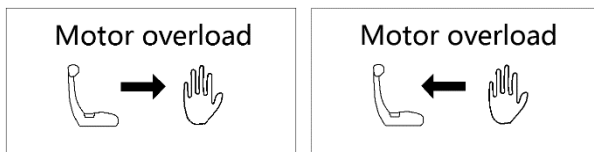
➤ **Front and rear motor (move forward and backward)**

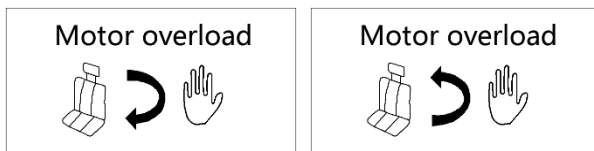➤ **Rotating motor (rotate inn and out)**

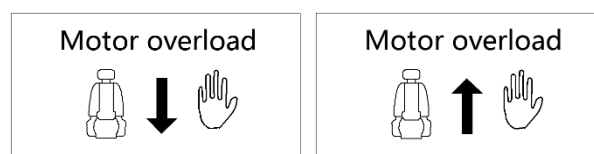➤ **Extending motor (extend out and return back)**


Possible reason:

1. The motor is blocked during running.
2. The seat is still locked.
3. Short circuit in motor.
4. Motor encoder is failed.


How to repair:

1. Remove the block;
2. Check seat locker and reset the seat program using manual mode. (refer to chapter 7.9);
3. Check if motor harness is damaged.

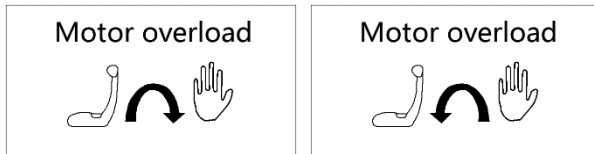➤ **Backrest motor (tilt forward and backward)**

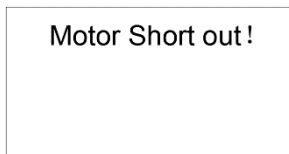13.3. Motor Short Circuit

Possible reason: motor or motor harness is damaged.

How to repair: Check motor harness or replace motor.


13.4. Motor Encoder Error

Motor encoder error means the controller cannot get the pulse from motor.

Remote control will display the running motor and it's moving direction.

Icons of encoder error for different motors:

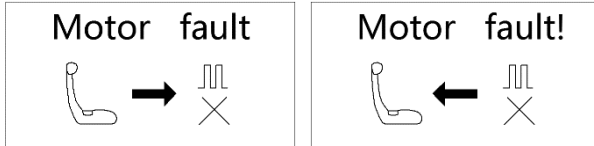
➤ **Front and rear motor encoder error (move forward and backward)**

Possible reasons:


1. The harness of motor is loose;
2. The harness of encoder is loose;

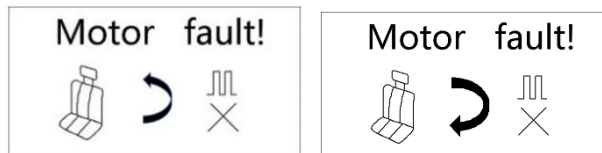➤ **Rotating motor encoder error (rotate in and out)**


How to repair: Check the harness of motor and encoder. Tighten it.

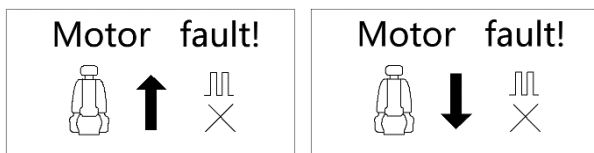
➤ **Extending motor encoder error (extend out and retract back)**

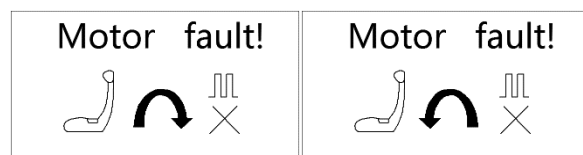➤ **Backrest motor encoder error (tilt forward and backward)**

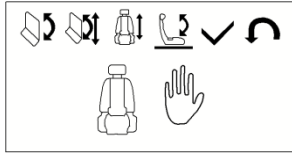13.5. Cannot extend out under programming mode

This error only happens under programming mode.

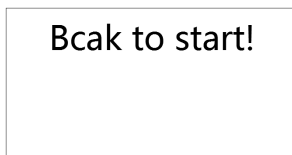
Possible reason:

You may try to extend this seat before rotate it more than 45 degrees.

How to repair:

when you program your seat, first rotate it more than 45 degree and then program to extend it.

13.6. Home Position Error

This error only happens when you enter programming mode or transmission mode.

Possible reason:

Your seat is corrected unsuccessfully when you enter programming mode or transmission mode (refer to chapter 8).

The sensor of programming position is damaged.

How to repair:

Re-enter manual mode to correct your seat.

Check the sensor of home position.

13.7. Low Voltage

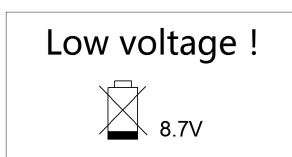
When your car battery voltage is less than 10V, the remote control will display this icon.

In this case, your seat cannot work until the battery voltage is over 10.5V.


The remote control will display battery voltage (refer to the right figure)


Possible reason: Battery voltage is too low.


How to repair: Charge your battery.

Attachment 1: electrical wiring diagram

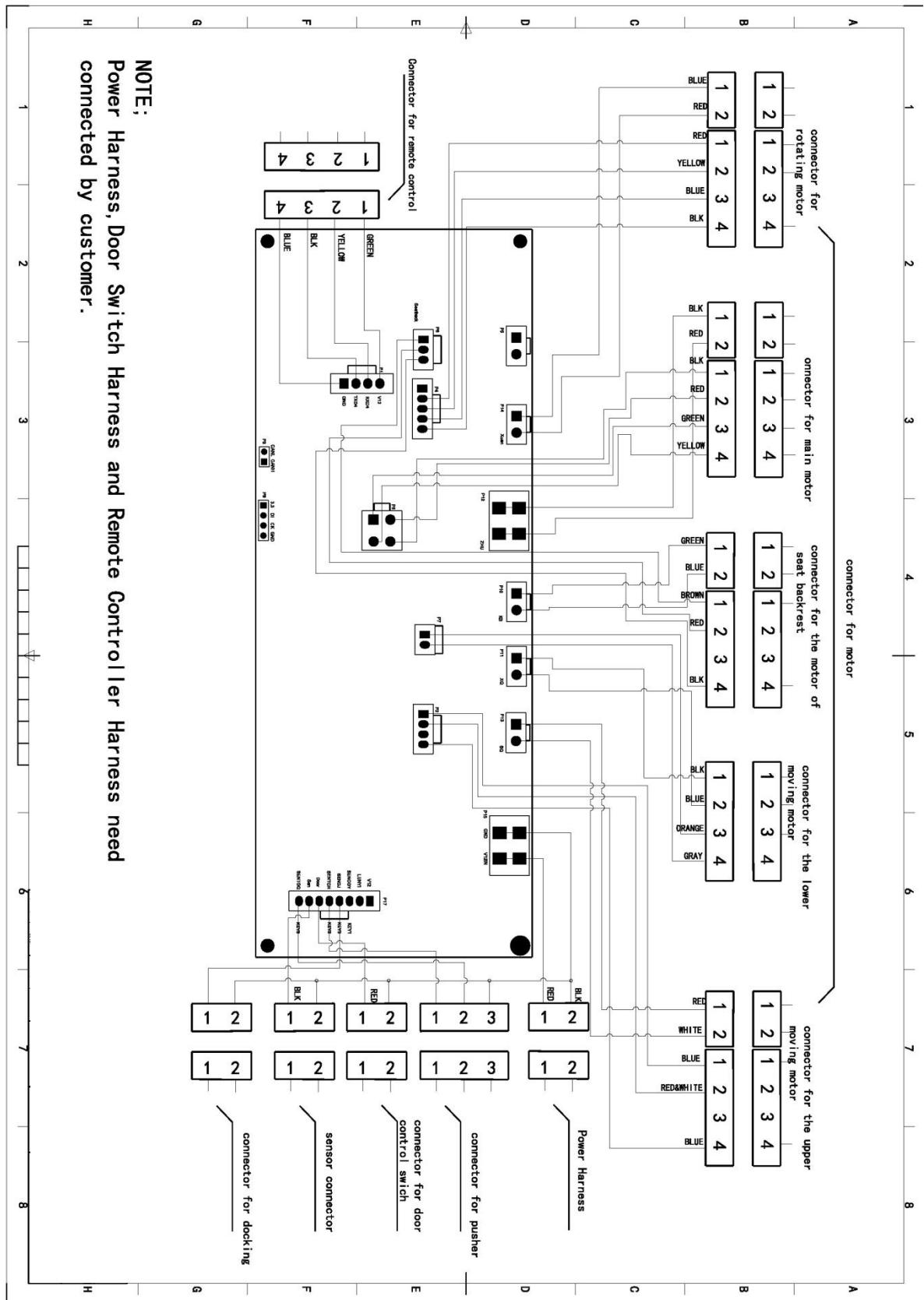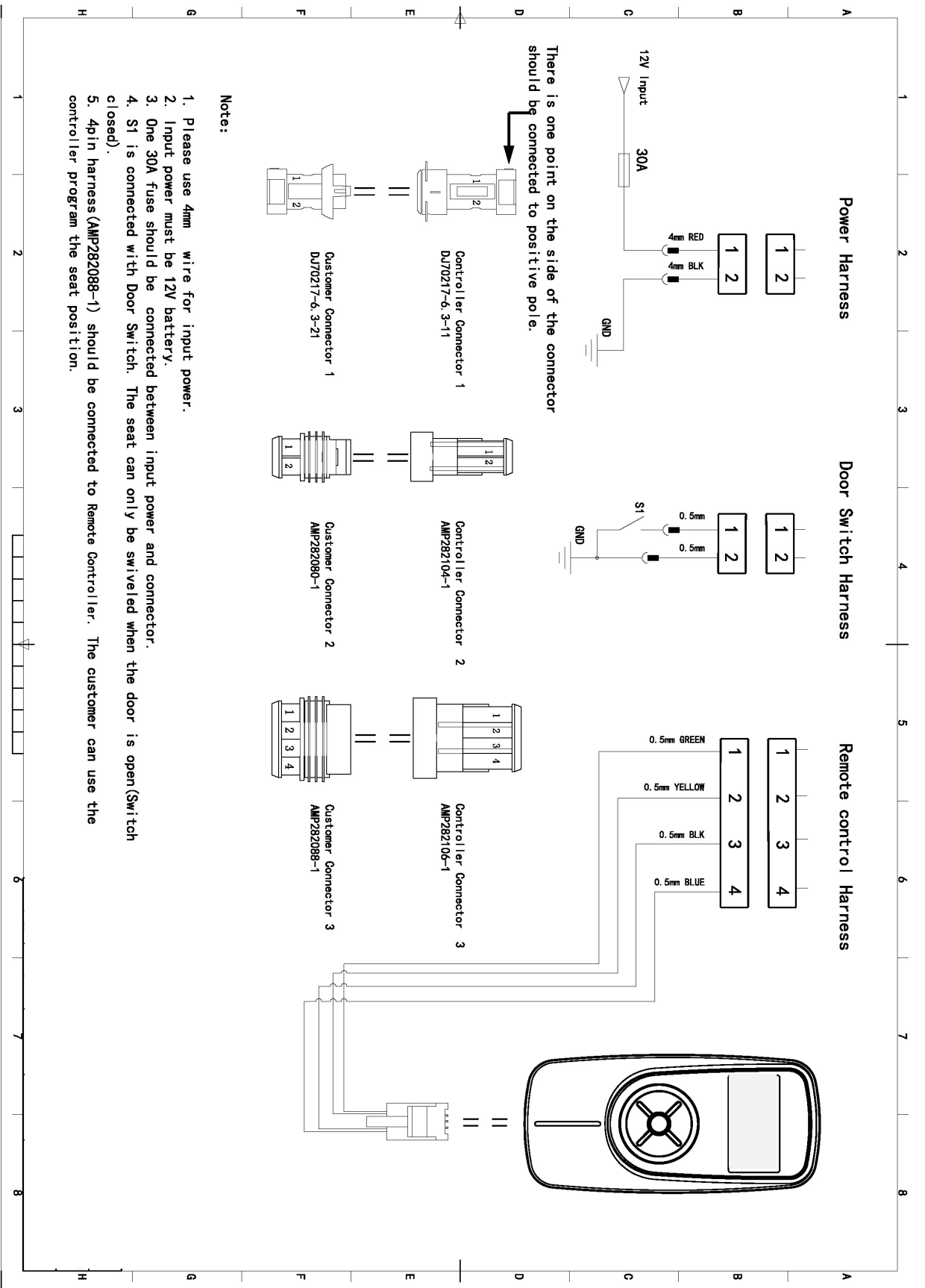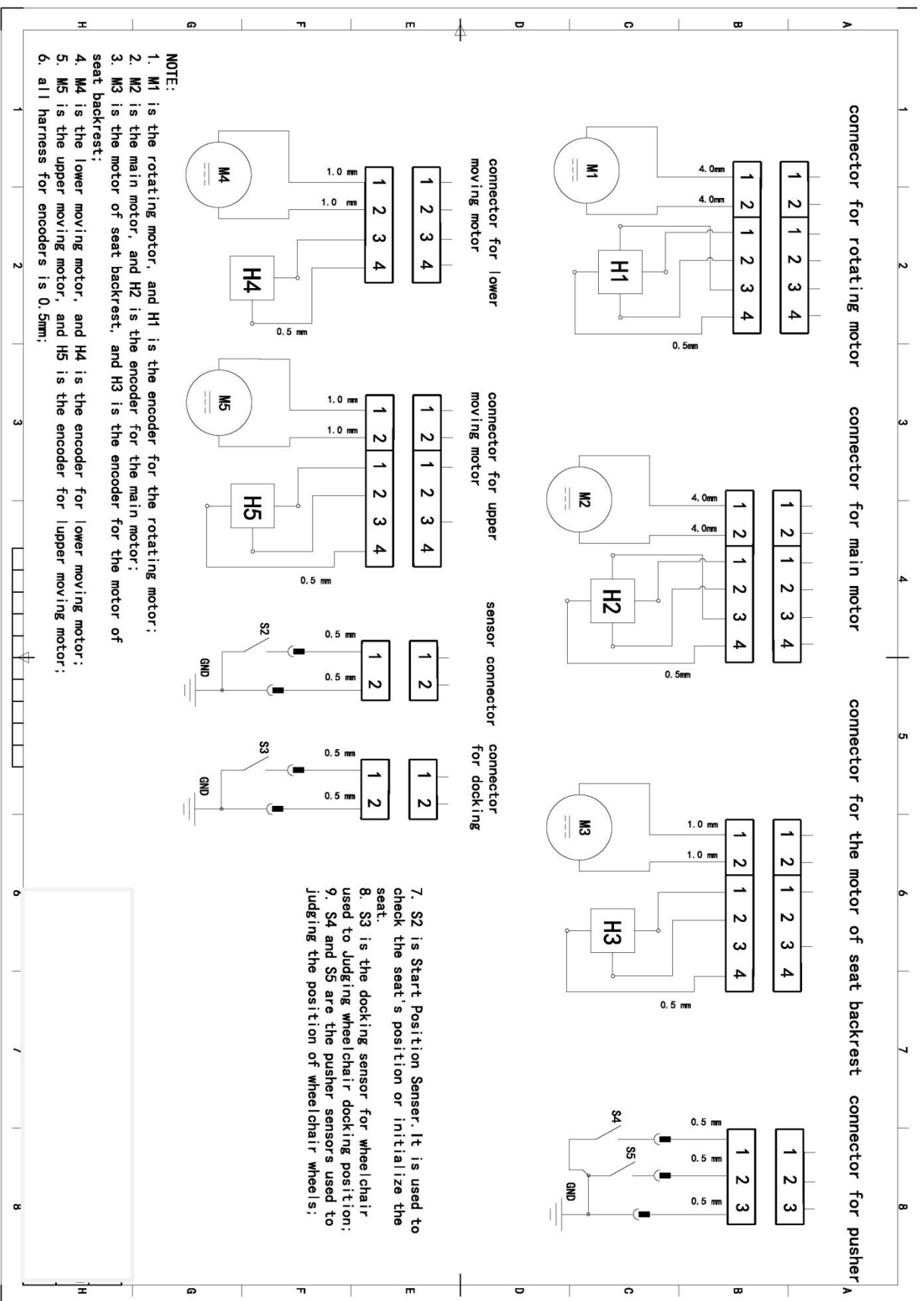NOTE:
 1. M1 is the rotating motor, and H1 is the encoder for the rotating motor;
 2. M2 is the main motor, and H2 is the encoder for the main motor;
 3. M3 is the motor of seat backrest, and H3 is the encoder for the motor of seat backrest;

4. M4 is the lower moving motor, and H4 is the encoder for lower moving motor;
 5. M5 is the upper moving motor, and H5 is the encoder for upper moving motor;
 6. all harness for encoders is 0.5mm;

7. S2 is Start Position Sensor. It is used to check the seat's position or initialize the seat.
 8. S3 is the docking sensor for wheelchair used to judging wheelchair docking position;
 9. S4 and S5 are the pusher sensors used to judging the position of wheelchair wheels;

GUIDOSIMPLEX srl - Via del Podere S. Giusto, 29 - 00166 Roma (Italy) Tel. +39 06.6152801 - Fax

+39 06.61528036 - Email infoclient@guidosimplex.it

www.guidosimplex.it